



# dry-sun uv 2

Electronic drying station for hearing aids.

INSTRUCTION FOR USE

EN	INSTRUCTION FOR USE	FI	KÄYTTÖOHJEET
DE	GEBRAUCHSANWEISUNG	SE	BRUKSANVISNING
FR	MANUEL D'UTILISATION	NO	INSTRUKSJONSHÅNDBOK
ES	MANUAL DE INSTRUCCIONES	HU	HASZNÁLATI UTASÍTÁS
PT	MANUAL DE INSTRUÇÃO	PL	INSTRUKCJA OBSŁUGI
IT	MANUALE D'USO	CZ	UŽIVATELSKÁ PŘÍRUČKA
NL	HANDLEIDING	SL	NAVODILA ZA UPORABO
TR	MANUALE D'ISTRUZIONI	BG	РЪКОВОДСТВО ЗА ЕКСПЛОАТАЦИЯ
GR	ΕΓΧΕΙΡΙΔΙΟ ΟΔΗΓΙΩΝ	RU	РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ
DK	BETJENINGSVEJLEDNING		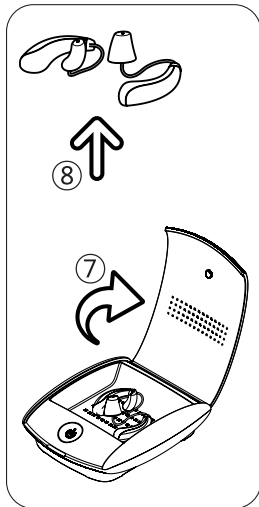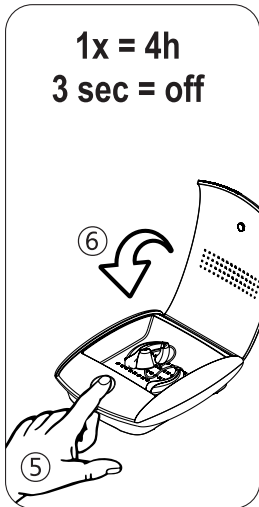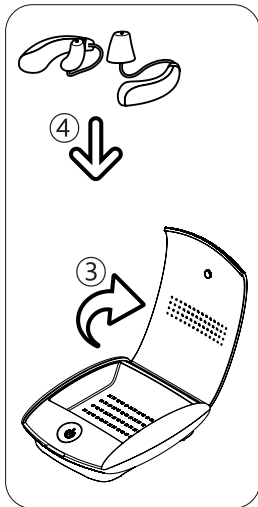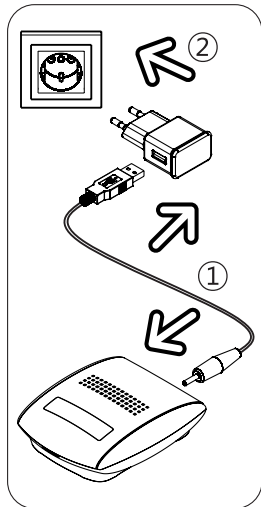

## EN INSTRUCTION MANUAL

### Dries and cares your hearing systems

Moisture and ear wax can impair the function of hearing systems and cause failures. The dry-sun uv 2 offers optimal care. Comfort drying at 50°C extends life time significantly. Ultraviolet light (253,7nm UV-C) eliminates 99,9% of germs and bacteria. This clinical cleanliness prevents ear infections risks and protects both your hearing systems as well as your health.

### Using first time

1. Insert the USB-cable into the 5 V power supply and the round plug connector into the jack on the rear side of your drying station
2. Then plug the power supply into an AC power socket ① The dry-sun uv 2 is ready to use

### Operation

3. Open the lid of your dry-sun uv 2
4. Place the with wet wipes pre-cleaned hearing systems with the battery-door open (if possible) into your drying station
5. Briefly touch the sensor button and the comfort drying program (drying + UV-C) is starting automatically
  - The 4 hours drying program with approx. 50°C warm air is fully processor controlled ① LED permanent light
6. Close the lid
  - Ultraviolet light (253,7 nm) eliminates germs and bacteria to 99.9% within 4 minutes ① LED blink light
  - After 4 min the UV-C light switch off automatically, the drying process continues ① LED permanent light
  - Within the last 4 minutes of the 4 hours drying program, a second UV-C hygiene runs ① LED blink light
  - After 4 hours the drying station automatically shutdown to stand-by ① LED switch off
7. Open the lid again
8. Remove the dried and hygienically cleaned hearing systems

## Maintenance and operation

- We recommend wet wipes for cleaning your dry-sun uv 2
- You can interrupt the drying process by holding the sensor button for more than 3 seconds
- The dry-sun uv 2 is very energy efficient; unplug the power supply only for prolonged absence

## Warnings

- Avoid contact with water
- Don't use the dry-sun uv 2 near basins etc.
- In the event of a malfunction, unplug the power supply and take the drying station to your retailer
- Don't disassemble the drying station and use only the supplied 5 V power adapter
- Children must be supervised to ensure that they don't use the drying station as a toy
- The drying station is not to be used by persons who have limited physical, sensory or mental abilities or who do not have sufficient experience and/or knowledge of the drying station, unless they are supervised for their own safety by a responsible person, or receive instructions on how to use the drying station
- CAUTION: Do not look into the UV-C light, as this can harm your eyes and your skin
- The drying station has an automatic room and temperature modulation as well as safety shutdowns, when the lid is opened during UV-C program
- Do not dispose of electrical appliances as unsorted municipal waste; use separate collection facilities
- Contact your local government for information regarding the collection systems available
- If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being

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© 2010-2021 flow-med GmbH. All rights reserved.
**flow-med GmbH**